
1.

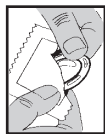
How to Use a Condom

Use a new condom for each act of sex.


1. Open the package carefully.

2.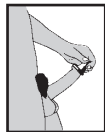


2. Before any sexual contact, place the condom on the head of the erect penis. **Gently press the air out** of the condom tip as you put it on. Air bubbles can cause condoms to break. Lesions, pre-ejaculate secretion, semen, vaginal secretions and blood can all transmit infectious organisms.

3.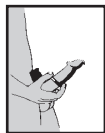

3. If you have a foreskin, pull it back before rolling on the condom. **Roll the condom down** so that it covers the whole hard penis. Gently press out air bubbles.

4.

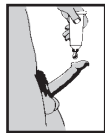4. You may want to add more water-based lubricant to the outside of the condom. If your partner is not wet enough, the condom can tear or pull off.

5.

5. After sex, while the penis is still hard, **hold the base of the condom firmly**. Pull out before the penis becomes soft. Try not to spill semen (cum). Throw the used condom away. Wash up with soap and water.

Caution: This product contains natural rubber latex which may cause allergic reactions.

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Kimono condoms are designed for the discriminating user. Sheer thinness and unparalleled feeling, are the hallmarks of Kimono brand condoms, and the traits that consumers have come to rely upon.

Kimono brand condoms are available in many retail outlets and can be ordered online at www.kimono-condoms.com.

Practice makes perfect. If using condoms seems awkward at first, consider trying them by yourself in private. As with anything new and unfamiliar, you may need to experiment and use condoms a few times before they become natural and easy.

Take care of your health...

Important Information: When used correctly every time you have sex, latex condoms greatly reduce, but do not eliminate, the risk of pregnancy and the risk of catching or spreading HIV, the virus that causes AIDS. Latex condoms can also reduce the risk of other sexually transmitted diseases (STDs), such as chlamydia and gonorrhea, that are spread to or from the penis by direct contact with the vagina and genital fluids.

Contact your doctor or pharmacist if you or your partner is using vaginal medicines that may come in contact with the condom. Some products may damage the condom.

Always get medical care if you think you have a sexually transmitted disease (STD). STDs do not go away by themselves.

Notes & Precautions

- If you use condoms the right way, they will work better.
- Use a new condom every time you have sex.
- Put the condom on **before** your penis goes into your partner's body.
- NEVER use lubricants with oil in them, such as vegetable or baby oil, hand lotion, or petroleum jelly, as these will

weaken latex condoms.

- If you want to use a sexual lubricant **ONLY** use **water-based** products such as Aqua Lube® (www.aqua-lube.com).
- Store condoms away from direct sunlight in a cool, dry place.
- If correctly and consistently used, latex condoms will help to reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted diseases.
- Single use - do not reuse condoms. Use a new condom every time you have sexual intercourse or other acts of sex between partners that involve contact with the penis.
- Latex condoms reduce the risk of transmitting STIs by providing a barrier against the source of the infection.
 - Latex condoms are most effective against STIs such as HIV infection (AIDS) and gonorrhea that are spread by contact with the head of the penis.
 - Latex condoms are less effective against STIs such as the Human Papillomavirus (HPV) and herpes. These STIs can also be spread by contact with infected skin that is not covered by the condom.
- If you have questions about condoms or if you believe you have an STI, contact a health care provider or your local public health clinic. In the USA call the National STD and AIDS Hotline at 1-800-342-2437 or 1-800-243-7889 (hearing impaired).

Manufactured for:
Mayer Laboratories, Inc.
1950 Addison Street
Berkeley, CA 94704, USA

EU Authorized Representative:
EZON.NL.b.v.
Koninginnegracht 81,
2514 AJ Den Haag, Nederland

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Pregnancy Rates for Birth Control Methods

(For One Year of Use)

The following table provides estimates of the percent of women likely to become pregnant while using a particular contraceptive method for one year. These estimates are based on a variety of studies.

“Typical Use” rates mean that the method either was *not always used correctly* or was *not used with every act of sexual intercourse* (e.g., sometimes forgot to take a birth control pill as directed and became pregnant), or was *used correctly but failed anyway*.

“Lowest Expected” rates mean that the method was *always used correctly with every act of sexual intercourse* but *failed anyway* (e.g., always took a birth control pill as directed but still became pregnant).

<i>Method</i>	<i>Typical Use Rate of Pregnancy</i>	<i>Lowest Expected Rate of Pregnancy</i>
Sterilization:		
Male Sterilization	0.15%	0.1%
Female Sterilization	0.5%	0.5%
Hormonal Methods:		
Implant (<i>Norplant™</i> and <i>Norplant™-2</i>)	0.05%	0.05%
Hormone Shot (<i>Depo-Provera™</i>)	3%	0.3%
Combined Pill (<i>Estragen/Progestin</i>)	8%	0.3%
Minipill (<i>Progestin only</i>)	8%	0.3%
Intrauterine Devices (IUDs):		
Copper T	0.8%	0.6%
Barrier Methods:		
Male Latex Condom ¹	15%	2%
Diaphragm ²	16%	6%
Vaginal Spermicide (<i>no previous births</i>) ³	16%	9%
Vaginal Sponge (<i>previous births</i>) ³	32%	20%
Female Condom	21%	5%
Spermicide:		
(<i>gel, foam, suppository, film</i>)	29%	18%
Natural Methods:		
Withdrawal	27%	4%
Natural Family Planning (<i>calendar, temperature, cervical mucus</i>)	25%	3-5%
No Method:	85%	85%

¹ Used Without Spermicide; ² Used With Spermicide; ³ Contains Spermicide

Data adapted from: Trussell J. Contraceptive Efficacy. In Hatcher RA, Trussell J, Stewart F, et al. Contraceptive Technology: Nineteenth Revised Edition. New York, NY: Ardent Media, 2007.

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